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
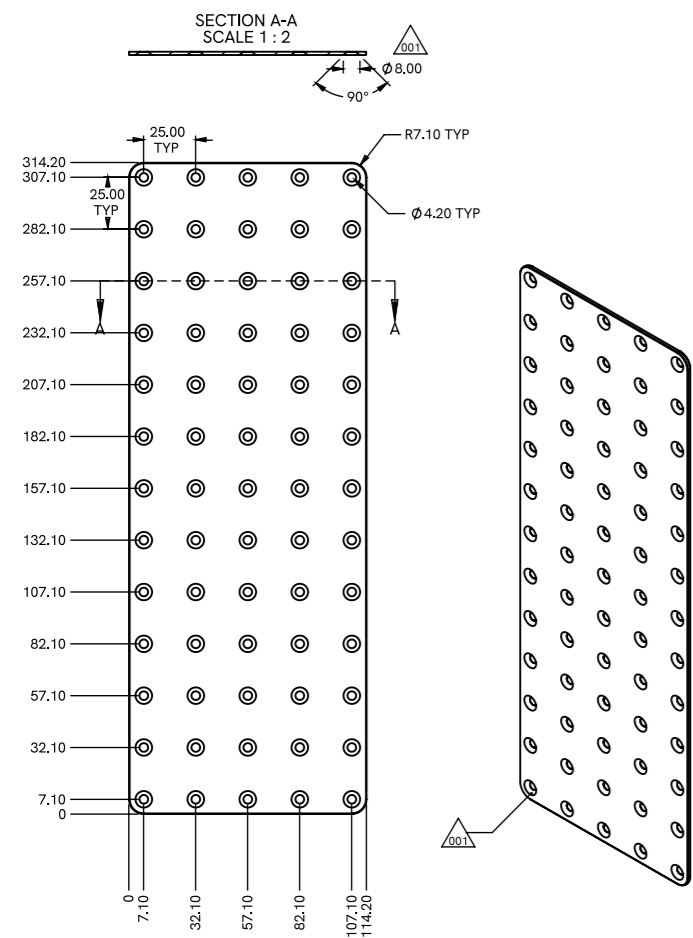
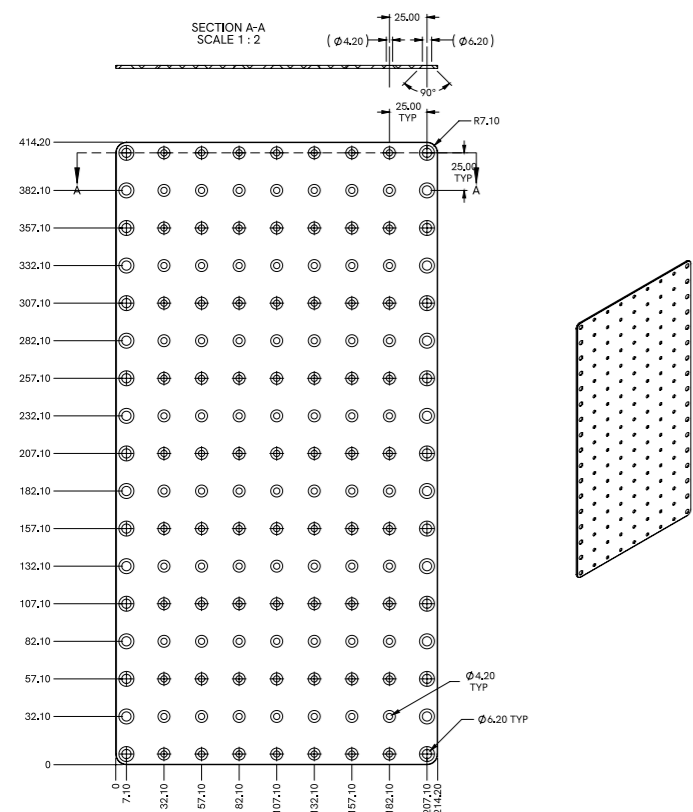
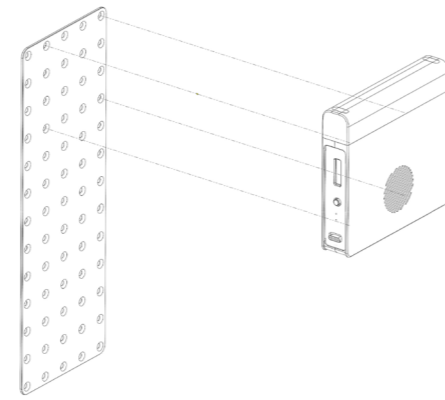
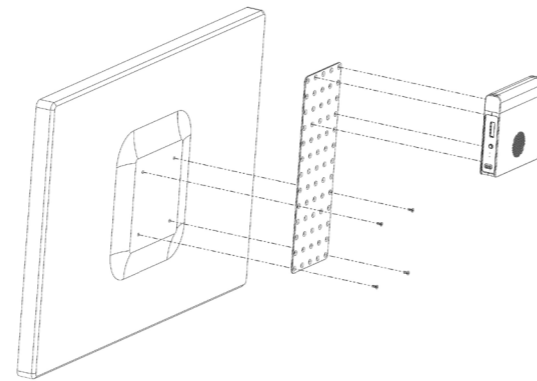
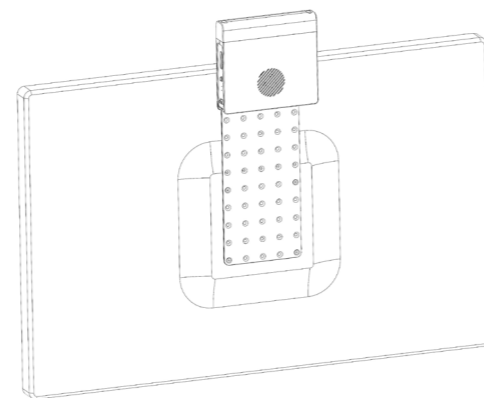
D

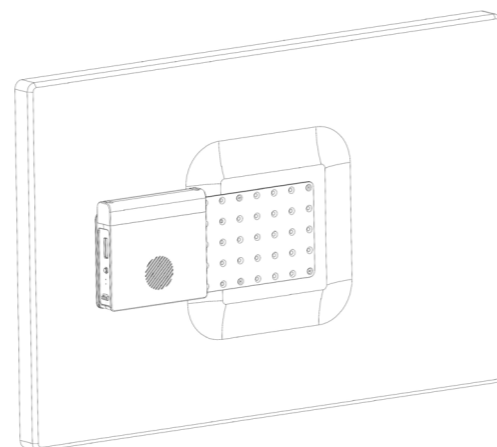VESA Plate Installation Guide

The VESA plate simplifies Falco product installation by mounting them directly onto standard monitors or devices using standard VESA mounts, eliminating the need for a dedicated location on the endoscope tower or next to the monitor.

Intended Use

The VESA plate is specifically designed for professional healthcare environments in facilities such as physician offices, dental offices, clinics, limited care facilities, freestanding surgical centers, freestanding birthing centers, multiple treatment facilities, hospitals, emergency rooms, patient rooms, intensive care units, and surgery rooms.

Its primary purpose is to securely mount Falco products on the back of monitors or onto various video sources and sinks, including VESA mounts.

System Specifications

	AMN_VESA_KIT01	AMN_VESA_KIT02
Weight	190.0 g	470.0 g
Nominal Thickness	2.0mm	2.0mm
Width	114.2 mm	214.2mm
Height	314.2	414.2mm
Material	Aluminum	Aluminum
Material Color	White	White

Equipment List

KIT 1	KIT 2
Part Number: AMN_VESA_KIT01	Part Number: AMN_VESA_KIT02
100 x 100 VESA plate	200 x 200 VESA plate
4x screws, M4, Flat Head, SS 18-8 DIN 7991	4x screws, M4, Flat Head, SS 18-8 DIN 7991
Installation Guide	Installation Guide

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Before beginning the installation process, make sure to plan it according to the specific size and shape of the monitor. While planning, take into account the positioning of the Falco device and the routing of the video cables and power cord.

We recommend installing the Falco device in a manner that ensures easy access to the front panel, control buttons, and OLED screen.

- Attach the Falco device to the VESA plate, using the M4 screws.  
**NOTE: The Falco device's recommended position is vertical (A).**
- Connect the plate to the back of the monitor (B, C). If the monitor is wall mounted or suspended from a boom, the plate can be mounted between the monitor and the monitor's mounting device.  
**NOTE: A horizontal mount is also acceptable (D).**
- To ensure proper mounting of the VESA plate between a monitor and a VESA mount, follow this guideline:

Use screws with a length (L) equal to the length specified in the monitor's manual (M), and then add an additional 2.0mm to account for the thickness of the VESA plate.

$$L = M + 2.0mm$$

WARNING:

- Using screws shorter than the instructed length to connect the VESA plate may lead to an insecure mounting of the monitor and could cause injury if the monitor falls from its mount.
- The VESA plate is exclusively designed for connecting Falco devices. It must not be utilized to connect any other devices.

# TERALDEK

## VESA PLATE INSTALLATION GUIDE